




Recommended Installation

The EOGas 4^{PLUS} Sterilizer should be located near an outside wall, and exhaust directly to the outside by means of $\frac{3}{4}$ " (1.9 cm) I.D. tubing and to minimize the impedance of the exhaust system.

Through-wall mounting kits are available from Andersen Sterilizers.


Note: Sterilizer must be placed in an area that can maintain 68°F or above room temperature and relative humidity should be controlled between 30% - 60% in all work areas.

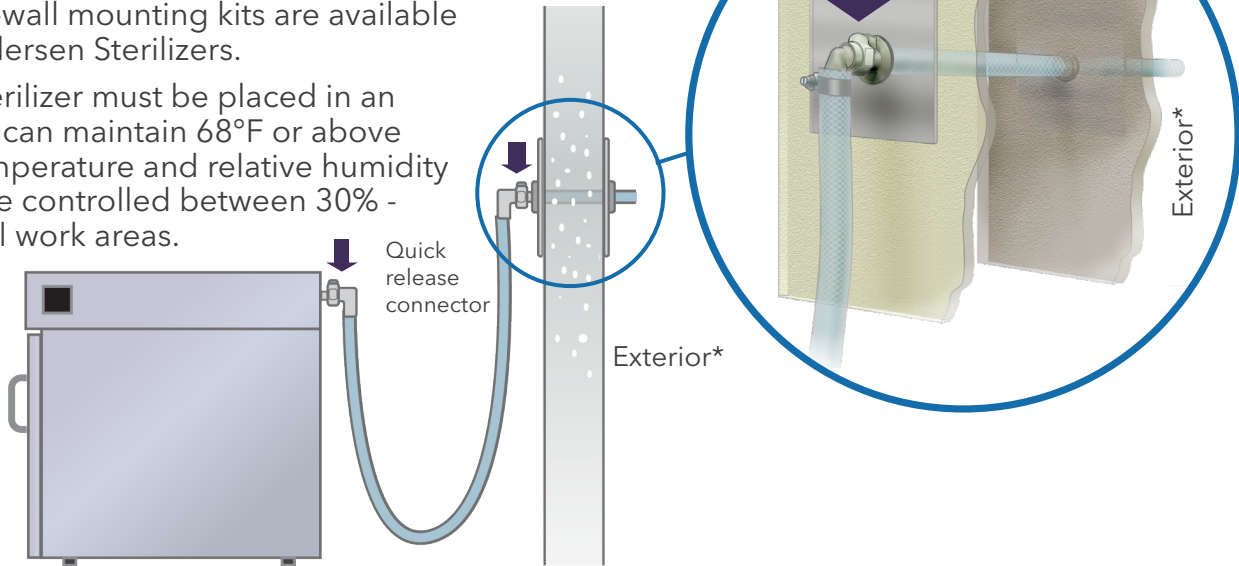

The EOGas 4^{PLUS} Sterilizer must be installed adjacent to an unswitched, non-GFCI, grounded power outlet that is dedicated to its exclusive use.

Alternate Option 1

Remote location installation

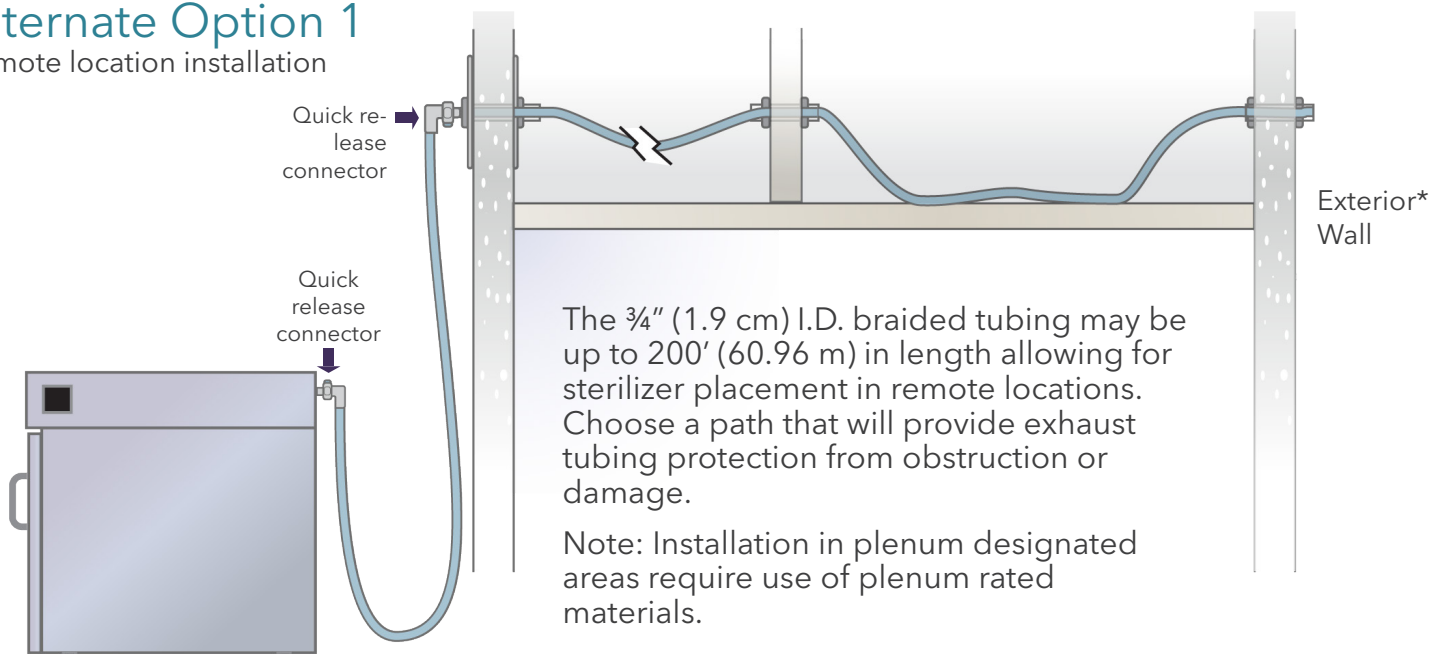


Alternate Option 2

Remote location installation

The through-wall mounting kit can also be used as a ceiling mount to allow eve or soffit termination.

Straight quick disconnect fittings are available from Andersen Sterilizers.

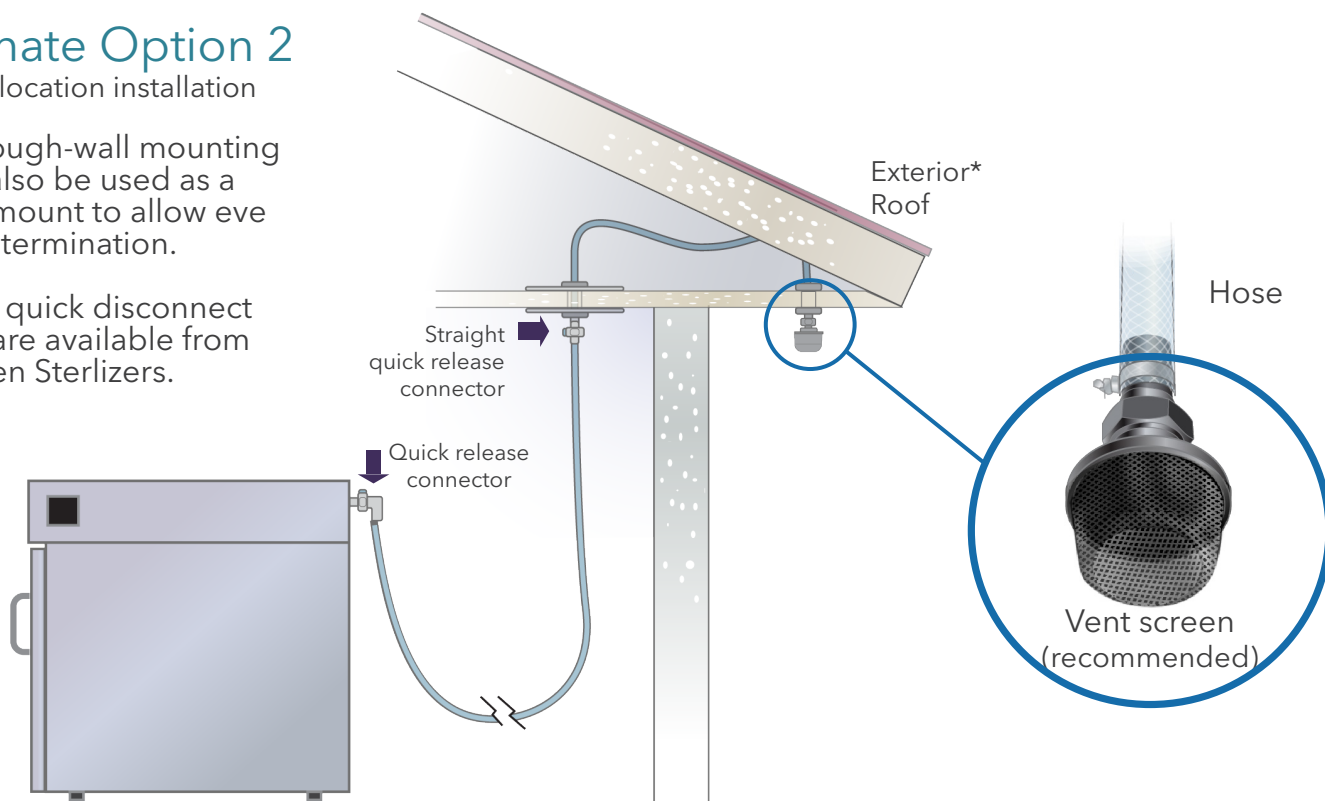

Important: Each EOGas 4^{PLUS} Sterilizer must have its own dedicated exhaust to outside air. The exhaust streams from multiple sterilizers may not be combined into a single exhaust stream that then vents to the outside.

All efforts must be made to eliminate any water intrusion or foreign objects from entering exhaust.*

Equipment Rating					
Model Ref.		Voltage	Hz	Power	Fuse Rating
AN4000.60	EOGas 4^{PLUS}	120v +/- 10%	60	630 Watts	8 Amp
AN4000.61	EOGas 4^{PLUS}	230v +/- 10%	50/60	580 Watts	4 Amp

***Please note:** AAMI guidelines recommend all EO sterilizers be vented through a dedicated exhaust line to the outside atmosphere directly or through an emission control system. The vent should not terminate within 25 feet of any building air intake source or near pedestrian or animal traffic, nor should it allow EO release into or to reenter the building.

For further installation assistance please call your Andersen representative or 1.800.523.1276